


TRI-GUARD


Cleaning sponge

Use to safely wash the endoscope during reprocessing.

Endoscope tip protector

Protects the tips of both large and small diameter endoscopes against damage reducing costly repairs.

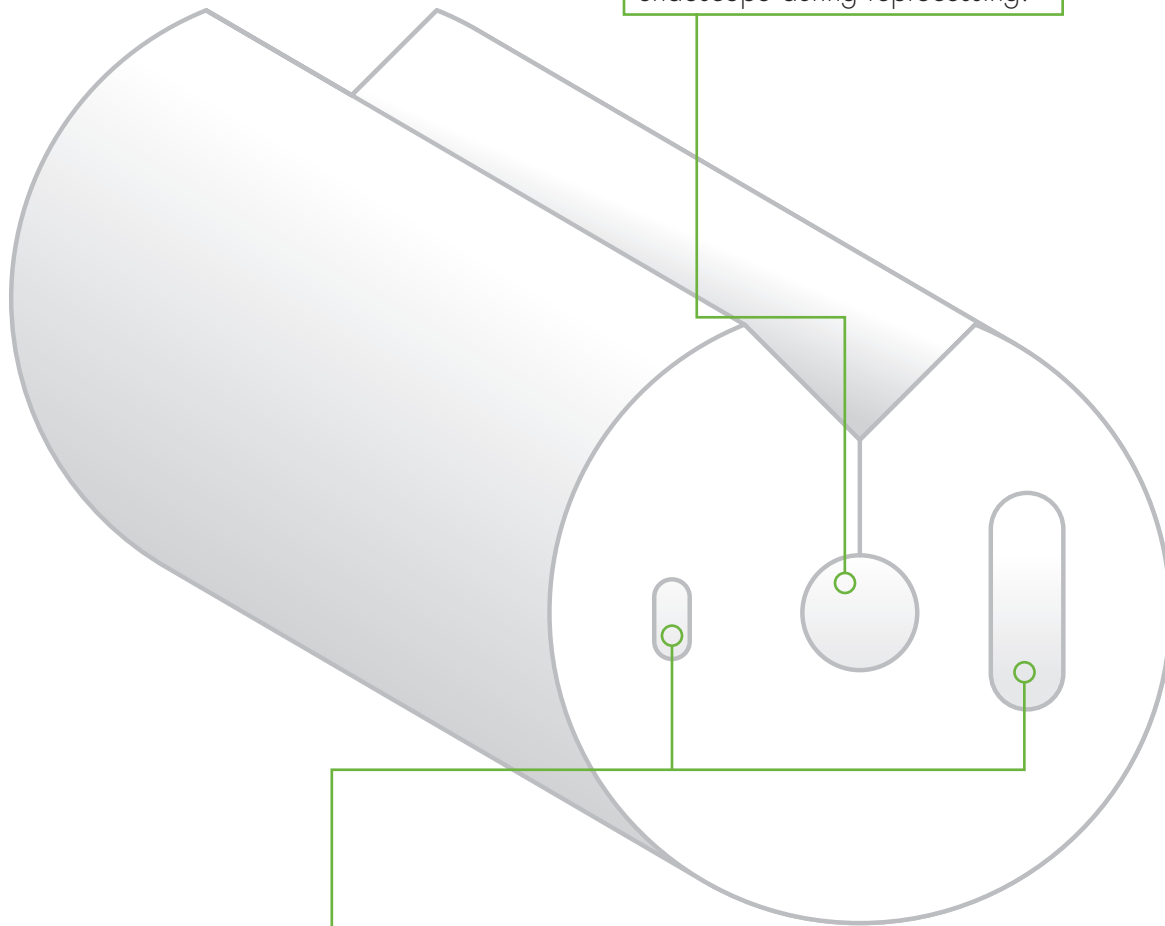

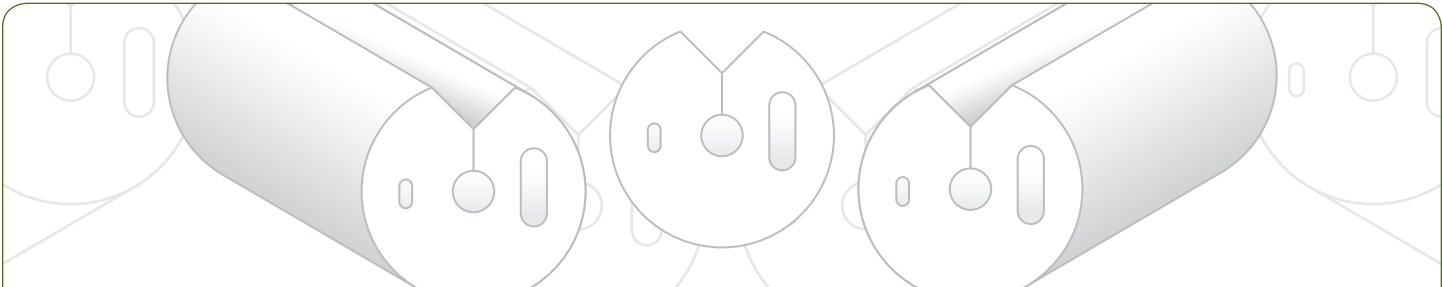

Tri-Guard Endoscope Tip Protector and Cleaning Sponge

Product Description

Endoscope Protection

Following the endoscope disinfection process, a clean Tri-Guard is placed onto the distal end of the endoscope using one of the two tip protector ports. The larger port protects most standard GI endoscopes, endoscopic ultrasound probes, ERCP and enteroscopes. The smaller port protects small diameter endoscopes such as bronchoscopes and intubation and ENT endoscopes. The endoscope is now ready for storage or transfer. The Tri-Guard should be kept in place until the procedure is about to begin.


Cleaning Sponge

The Tri-Guard central core can be used to wash the endoscope sheath and all external features using appropriate cleaning solutions.

Single Use Only. Non-Sterile. US Patent Pending.

For Pricing Call 877-572-9985

Features	Benefits
Endoscope tip protector	Protects the tip of the endoscope against damage reducing costly repairs.
Cleaning sponge	Use to safely wash the endoscope during reprocessing.

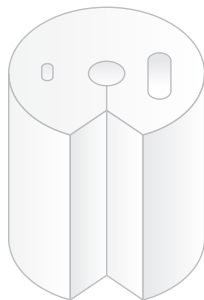

Product Order Number

CC-TRI

Product Description

The Tri-Guard endoscope device consists of a one-piece, disposable sponge with two ports and a central core. It is used as an endoscope tip protector and an endoscope sponge.